

AIS QuickStart Guide

How to View Exams Uploaded to AIS


1 Open up Internet Explorer on the desktop and go to the following website:
 www.antechimaging.com

2 Enter your User Name and Password and click Login.
 User Name = xxxxxxxx
 Password = xxxxxxxx.




* There is a "forgot username and/or password" option if needed.

3 To find a study, use the drop down menu titled "From Study Date" and enter "Date Range" to locate the patient you are looking for. Be sure to populate the dates in field "To Study Date".

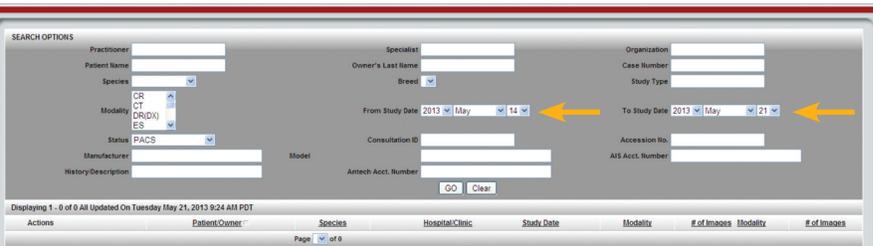

Displaying Consults

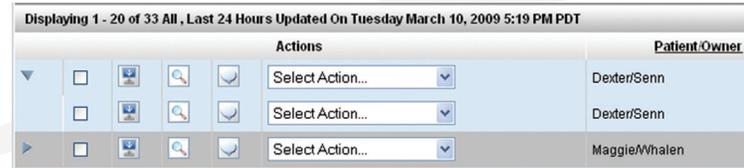

-  This icon indicates that there is more than one consult for a patient.
-  Click on the icon and it will turn down to and list the additional linked consults from the most recent to the oldest.
- This icon, when checked, allows for multiple studies for a patient to be displayed.
- This box can be checked along with these icons    for multiples of these commands to be viewed.
-  This icon will download the images for this case. Checking the box and then clicking on the icon will download multiple studies for this patient.
-  This icon displays the single study for this patient in a separate viewer. Checking the box and then clicking on the icon will display multiple studies.
-  This icon is used for submitting a consult for interpretation by a boarded specialist.

YOUR IMAGE IS OUR BUSINESS.



a VCA ANTECH company


800.268.5354 soundeklin.com

AIS QuickStart Guide

How to View Exams Uploaded to AIS

Actions

Select pull down to change an action

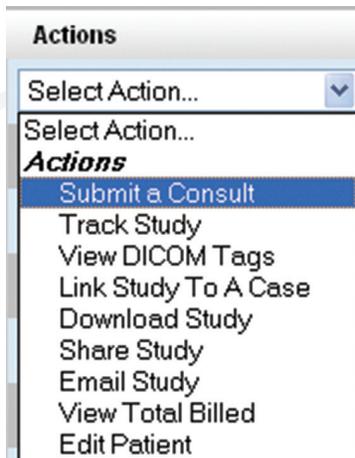

Select Action Definitions

Submit a Consult

Used for submitting a consult for interpretation by a boarded specialist.

Track Study

Tracks everything done to the medical record.

View DICOM Tags

This icon displays the DICOM tags associated with the images for this study.

Link Study To A Case

To attach, "Awaiting Media" case(s) to an existing PACS.

Download Study

This icon will download the images for this case. Checking the box and then clicking on the icon will download multiple studies for this patient.

Share Study

Allows the case to be shared with another specified rDVM. The timeframe that the case is allowed to be accessed can be chosen from a date range you choose (default is 1 month) or you can share indefinitely by selecting a check box.

Email Study

Allows you to email a study with a direct link to anyone with an email address.

View Total Billed

Shows your total storage and cost of your cases.

Edit Patient

Allows you to edit Patient information.

Additional AIS Logins

To setup additional AIS logins (for referral purposes), please call AIS at 877.727.6800, option 1.

YOUR IMAGE IS OUR BUSINESS.



a VCA ANTECH company

800.268.5354 soundeklin.com