

LAMP REPLACEMENT INSTRUCTIONS:

Collimator lamp specifications:

FCS type, 24 VAC, 150 watt

May be purchased from:

Most commercial photography (camera) stores or Premium Quality Lighting,
Part # 82262, 1-800-323-8107, www.pqlighting.com

WARNING

1. Turn off power prior to removing any covers.
2. Do not look directly at the lamp during operation.
3. Do not make exposures when the lamp cover is not in place.
4. Do not touch the new lamp with bare fingers at any time, as skin oils on the lamp will reduce lamp life. Use the plastic wrapper to hold the lamp.
5. Be careful not to disturb the lamp bracket position when removing or inserting the bulb as this will affect light field alignment (as shown in Figure 4).

1. Press the two rear cover button locks in and remove the rear cover as shown in Figure 2.
2. Remove the old lamp and replace it with the new one, making sure the lamp pins are pushed completely into the socket as shown in Figure 3.
3. Replace the rear cover, making sure the button locks snap into the holes in the cover.

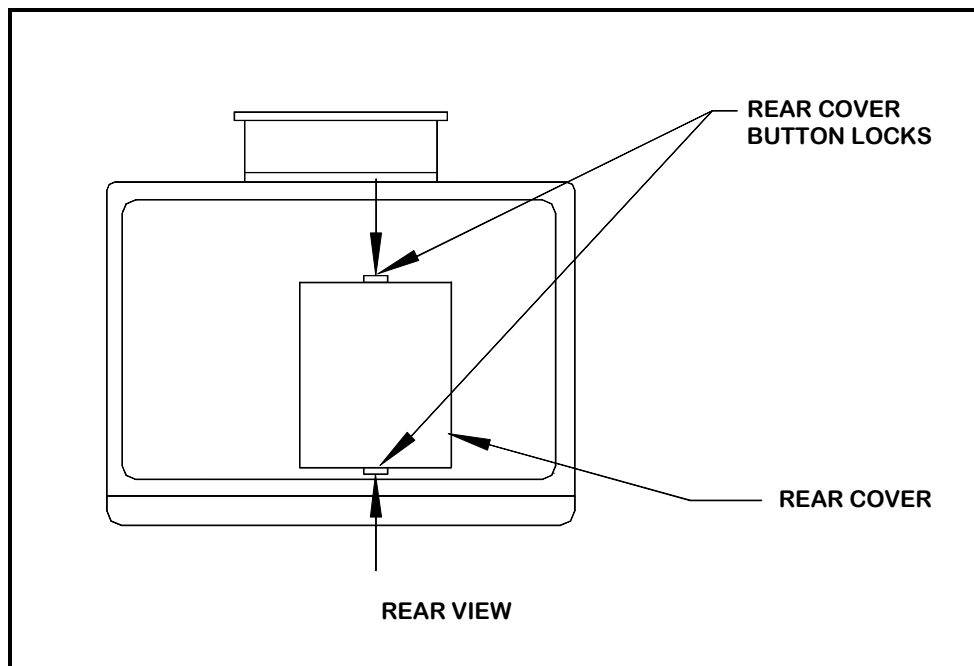


Figure 2 - Rear cover removal

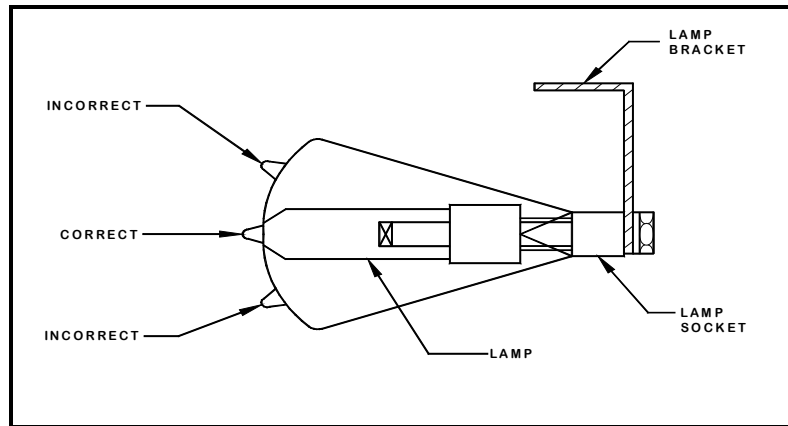


Figure 3 - Lamp replacement

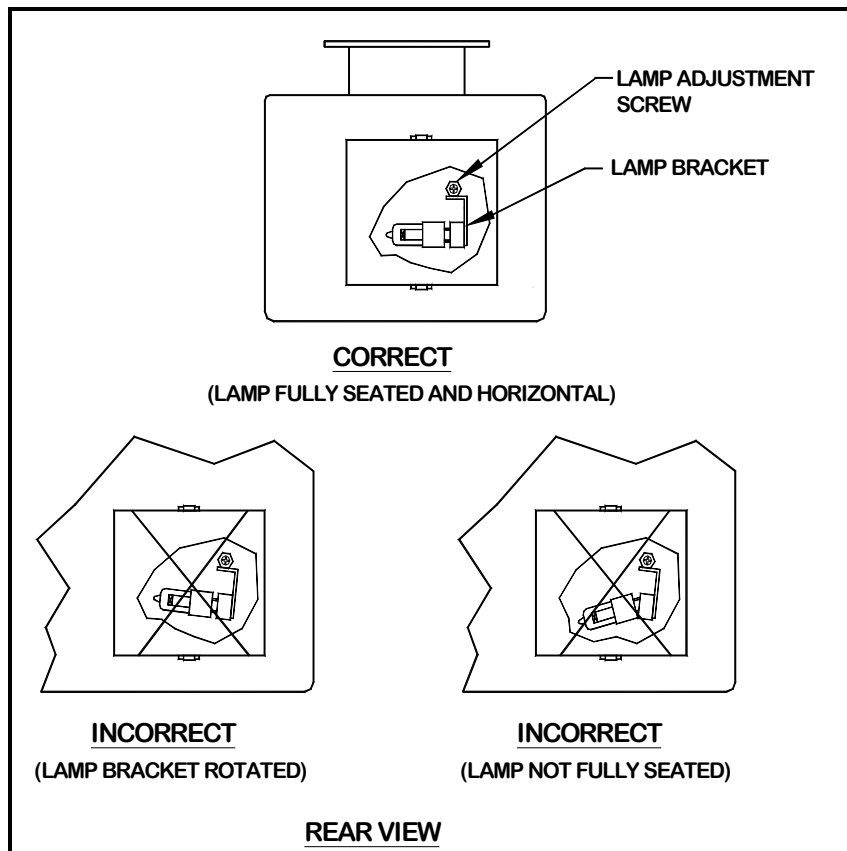


Figure 4 - Lamp and Bracket Positioning