

SMART DR™

Quickstart

1

Add a Patient

Click the Add button



2

Patient Information

Enter the patient information in the appropriate fields.

Required information is in red.

Add Patient Information				Add Owner Information	
Patient Name: ROVER	Weight: 55	Units: lb	Species: Canine	Owner Last Name: SMITH	
Patient ID: 0713201413579	DOB: 7	Years: Months:	Breed: LABRADOR RETRIEVER	Owner First Name:	
Secondary Patient ID:	Sex: Male			Email Address:	

3

Shot List

Create a list of views to acquire by selecting the desired Region, Anatomy, and View Positions. As the view positions are selected they will populate to the shotlist on the right.

Select Region	Select Anatomy	Select Shots
<ul style="list-style-type: none"> Body Left Forelimb Left Hindlimb Right Forelimb Right Hindlimb 	<ul style="list-style-type: none"> Abdomen Spine Pelvis C spine Skull Thorax Whole Body 	<ul style="list-style-type: none"> OV Lat Left Lat Right VD VD OBI Right VD OBI Left

ROVER SMITH
ID: 0713201413579
SMITH

Study Description:

Shot List

- Body Abdomen Lat Right
- Body Abdomen VD

Click the Next button.



4

Edit Views

Set the X-Ray generator to the appropriate value as indicated on the technique chart.

Tap the Retake button to Retake a view.



Tap the Add button to add additional views.



5

End Study

Once you have acquired all of the desired views click the End Study button to end the study and automatically send the images to the PACS server.



YOUR IMAGE IS OUR BUSINESS.



800.819.5538 soundvet.com
resources.soundvet.com