

# cSeries™ Quickstart

## 1 Patient Information

Enter Patient Information:  
LASTNAME PATIENT NAME  
(i.e. Smith\_Fuzzi), Date of  
birth, Unique patient ID, Sex.

The screenshot shows a form with the following fields: EXAM (Manual), PAST (Pending List), Name (Fuzzi Smith), DoB (01/01/2000), Age (12 Years), ID (1234), and Sex (Male selected).

## 2 Select Set





## 3 Create Shot

Create a shot list using the  
anatomy and view options.

The screenshot shows a patient information summary (Smith Fuzzi, DoB: 1/1/2008, Age: 4 Years, ID: 1234, Sex: Male) and a list of shot options: R Lat Tha, R Lat Abd, YD Abd, and YD Tha. The 'YD Tha' option is selected.

## 4 Start Exam

Select Start Exam.  
Prepare Patient.  
When system is READY  
make the first exposure.

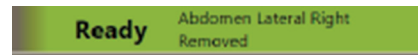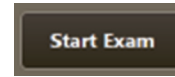


## 5 Evaluate Image

Evaluate the image for Motion,  
Positioning, Image Orientation  
and Diagnostic Quality.



## 6 Retake

Select RETAKE to retake the  
view. If the image is good,  
continue with exposing the  
next view in the list.

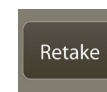

## 7 Image Orientation

To change the image orientation  
utilize the flip and rotate tools,  
the image will save as it is.




## 8 Refining the Study

Repeat steps 4 - 7 as needed,  
until study is complete.

## 9 End Exam

When finished select  
END EXAM

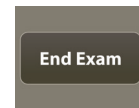

YOUR IMAGE IS OUR BUSINESS.



a VCA ANTECH company

800.268.5354 soundeklin.com