

NE Software Tools



Rotate Left

Rotate image 90 degrees to the left.



Rotate Right

Rotate image 90 degrees to the right.



Flip Horizontal

Flip image left to right.



Flip Vertical

Flip image top to bottom.



Reposition

Reposition the image in the viewing window, once it is zoomed. (Use in conjunction with the Zoom tool.)



Zoom In

Zoom into image.



Zoom Out

Zoom out of image.



Full Screen

Return Image to full screen.



'R' Marker

Apply Right marker.




'L' Marker

Apply Left marker.




Crop

Trim image margins.

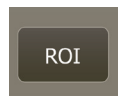


ROI

Adjust processing Region of Interest.

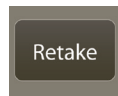


Retake

Retake image onscreen. Select Restore to cancel Retake. Note: you can only keep 1 retake per view.

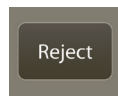

Reject

Reject or Delete Image onscreen. Select Restore to cancel Reject.

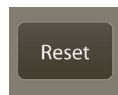

Reset

Reset all changes back to original image.




Undo

Undo the last 2 changes made to an image.




Annotation

Add user annotations to an image.

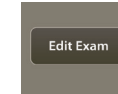


Edit Exam

Return to anatomy/ views to edit shot list.

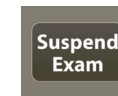


Suspend Exam

Pause current patient exam. Resume exam from Pending list.

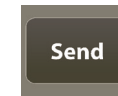

Send

Finalize and send current images for review. Images can no longer be manipulated in capture.

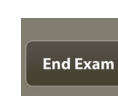

End Exam

Finalize and send current exam for review. No images can be added to the study nor can the study be manipulated.

YOUR IMAGE IS OUR BUSINESS.



a VCA ANTECH company

800.268.5354 soundeklin.com