

# tru | DR™ cSERIES

## Time (seconds) in Fractions

Centimeters	1-10	11-15	16-25	26-30
mA	300	300	300	300
kVp	85	95	110	110
Time (seconds)	1/120	1/120	1/120	1/60
mAs	2.5	2.5	2.5	5

## Time (seconds) in Decimals

Centimeters	1-10	11-15	16-25	26-30
mA	300	300	300	300
kVp	85	95	110	110
Time (seconds)	.008	.008	.008	.016
mAs	2.5	2.5	2.5	5

Note:

For patients over 30cm, double the mAs  
DIAGNOSTIC REX VALUE 300+

\* Due to variability between x-ray generators, optimal techniques may vary somewhat from the values printed above.



(800) 819.5538  
resources.soundvet.com  
soundvet.com

