

# Quick Guide for Sound HD Dental Software (Storage/Worklist Configuration)

## STORAGE CONFIGURATION

1. Press the setting button at the top (Figure 1) and press Network from the Setting window then click Storage (Figure 2)

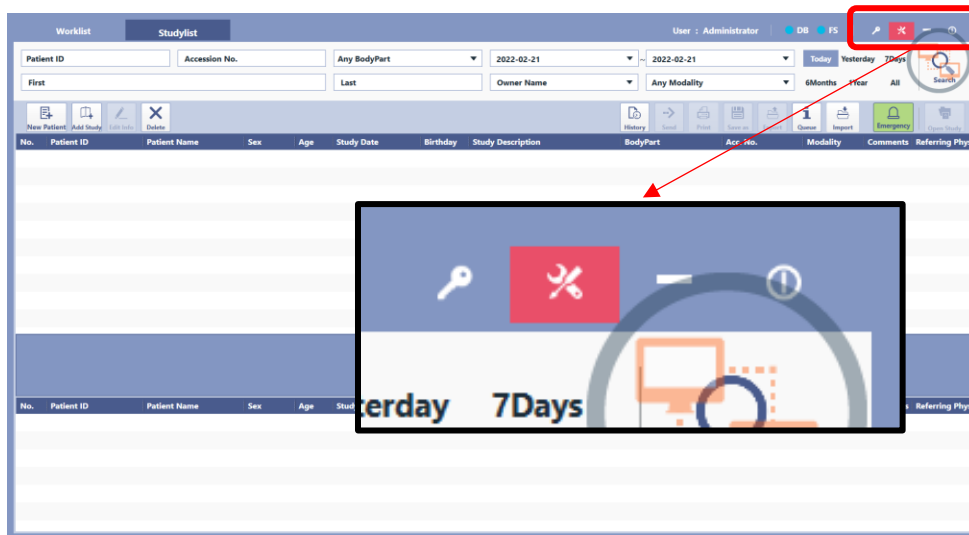


Figure 1. Setting Button from Studylist/Worklist view

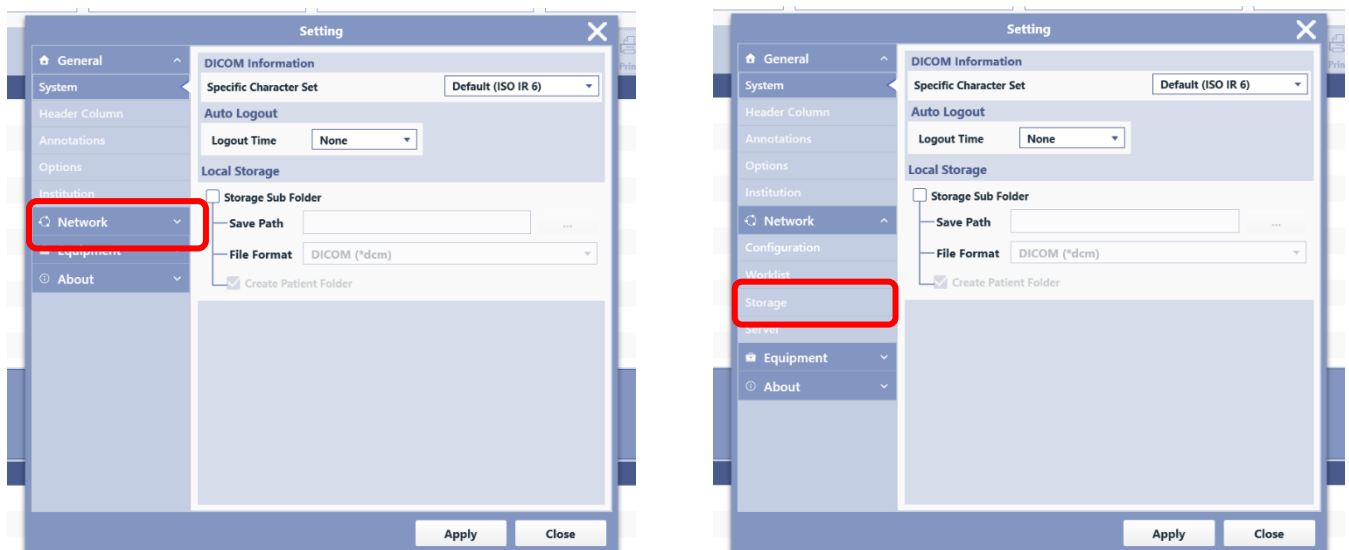


Figure 2. Network and Storage in the Setting Menu

2. Press Add button (Figure 3) and add AE Title, IP Address and Port to the Server Information and click Add (Figure 4)

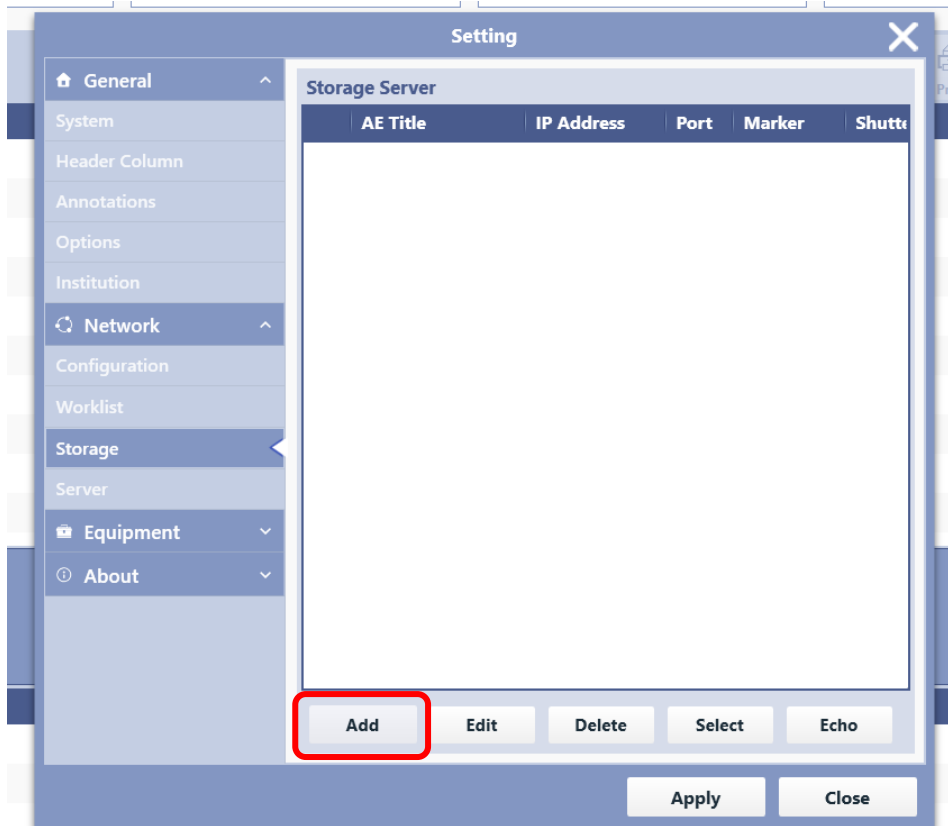


Figure 3. Adding a Storage Server

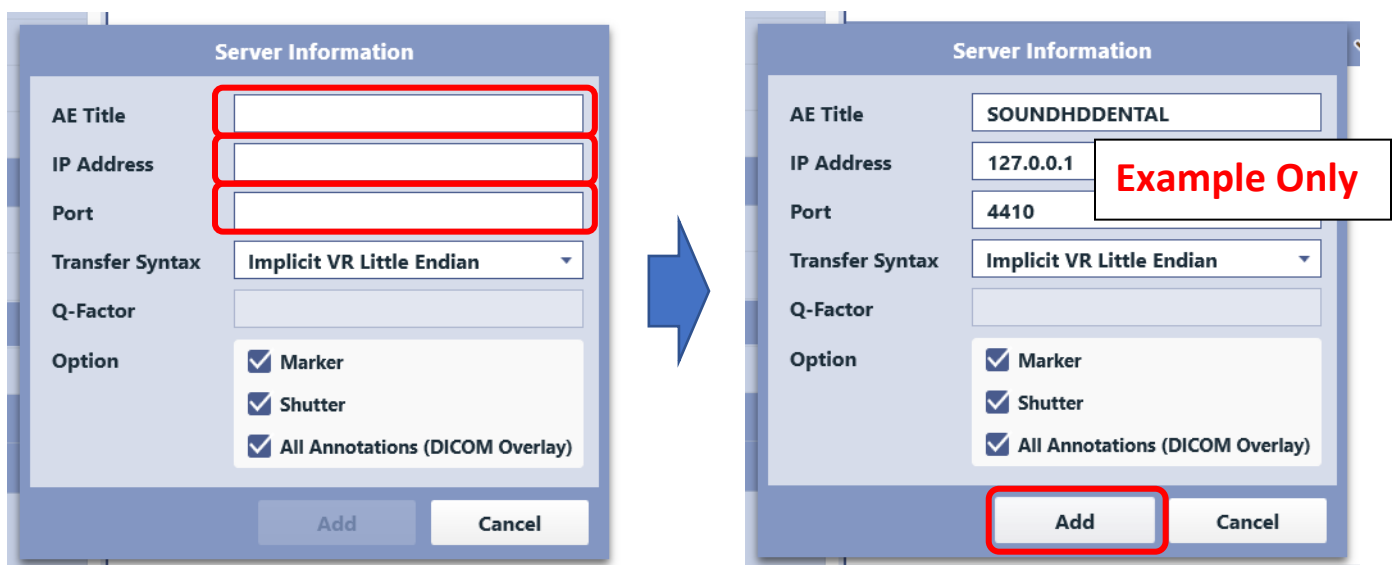


Figure 4. Entering AE Title, IP Address and Port number

3. Once completed with adding Storage server, click Echo to verify DICOM connectivity with server. If the server information is not correct or the port is blocked from the server, it will say “Connection Failure”.

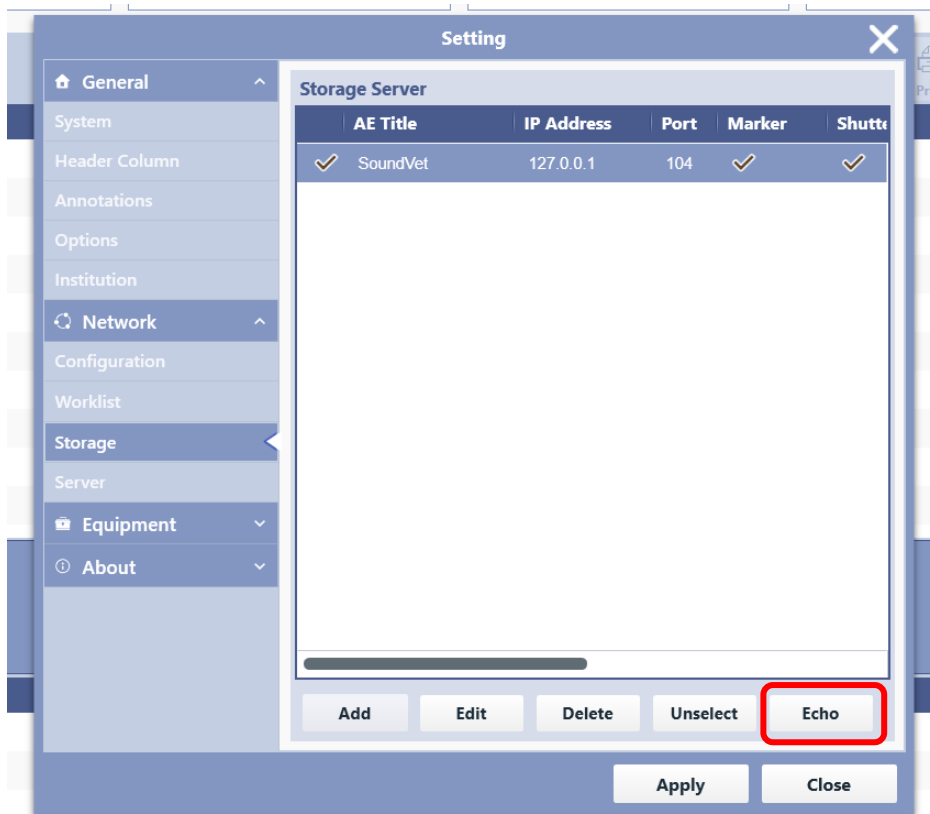


Figure 5. DICOM Connection verification of Storage Server


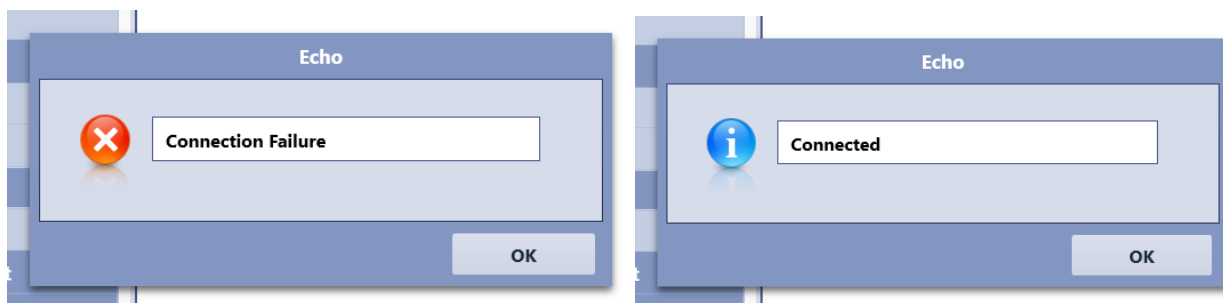



Figure 6. Example of Echo Test Result

4. You can add more than one server and have them selected to send the study to all the servers.

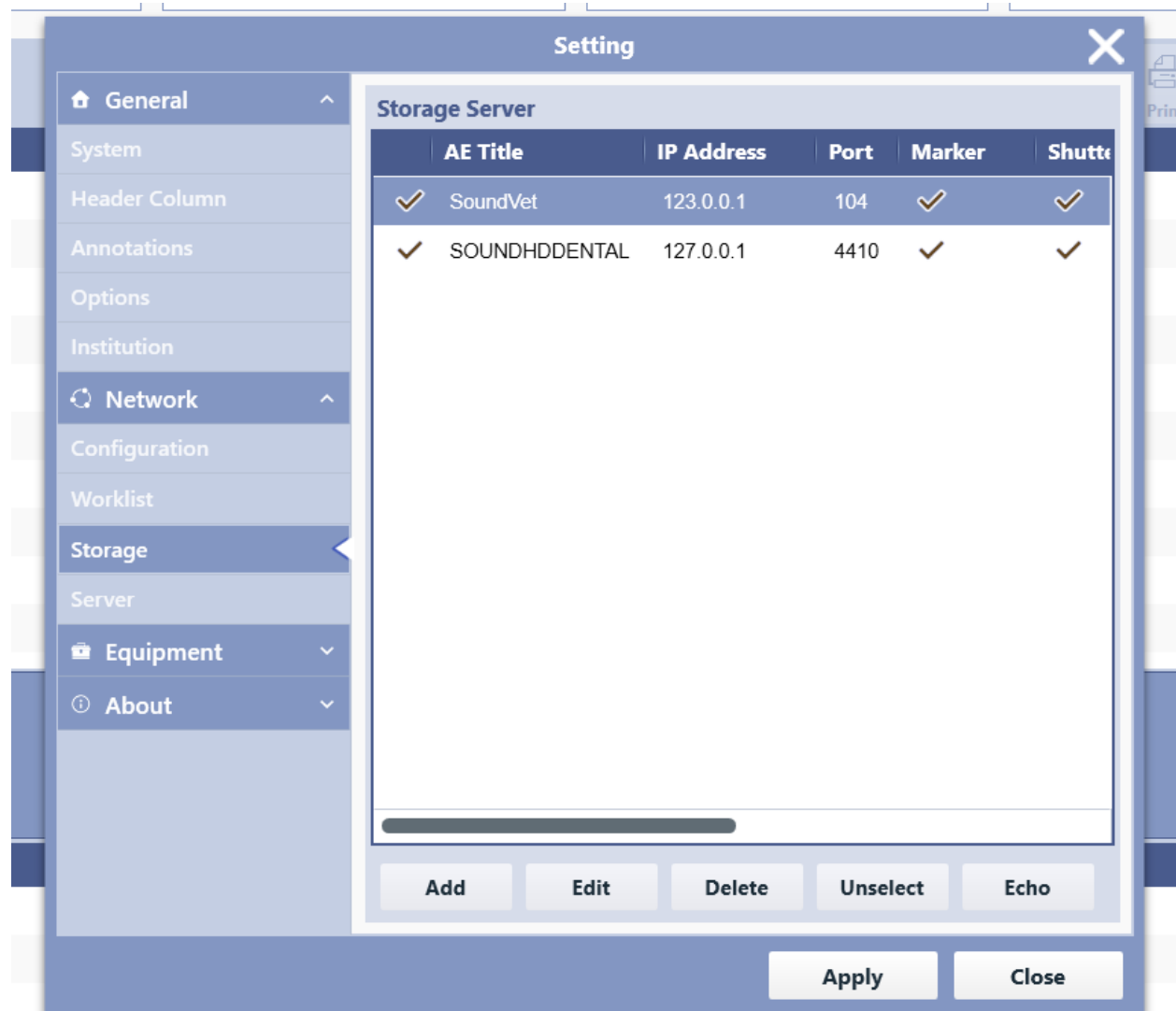

## WORKLIST CONFIGURATION

1. Below is where you can configure Worklist Server and the steps are the same as configuring storage server above. Follow the steps of Storage Configuration.

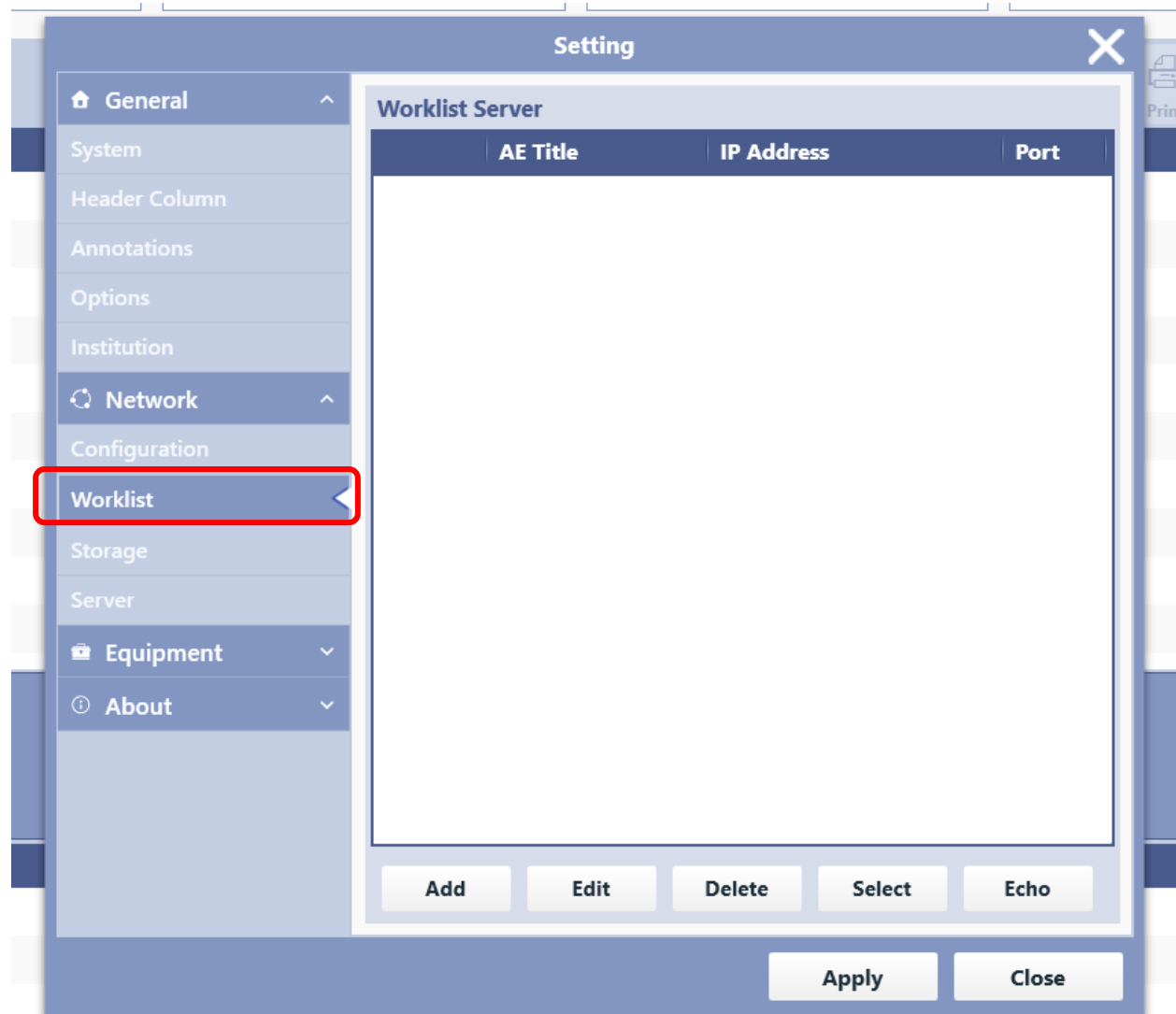

## SEND OPTIONS

1. If you want to automatically send images to Storage Server when a study completes, click “DICOM send when a closing session”.

